


acrytemp



BRILLIANT CHARME

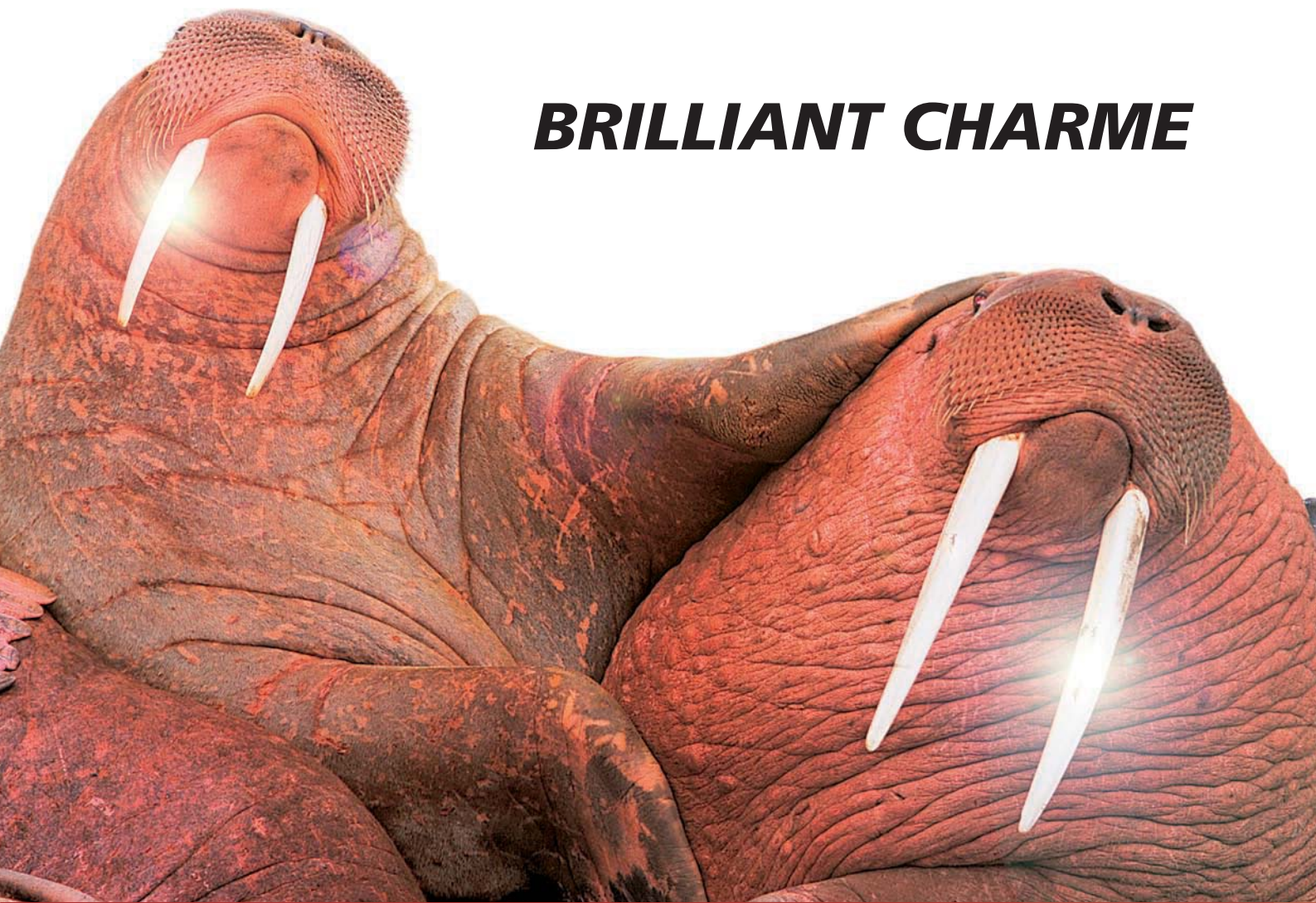

Clinical

Temporary C&B Composite

Zhermack[®]

CLINICAL



acrytemp

Natural shine for incomparable aesthetic

CLINICAL APPLICATION

ZHERMACK presents the self-curing composite for *fast preparation in the surgery* of temporary crowns and bridges.

UNIQUE ADVANTAGES

- **Protection** of the preparation
- Respect of the **dental-parodontal relationship**
- **Maintenance** of the dental position
- High **breakage resistance**
- **Natural aesthetic finish** of the temporary crowns and bridges
- **Optimum sealing** of the edges
- Creation of **compact and smooth surfaces** that reduce the risk of plaque accumulation
- **Low curing temperature** to prevent irritation of the tissue



Repositioning of the impression on the preparation



Finishing of the temporary crowns and bridges in ACRYTEMP

tics!



1 Application of ACRYTEMP inside the impression taken before working on the preparation



3 Removal of the temporary crowns and bridges from the impression after setting



5 Temporary crowns and bridges



OPTIMAL CHARACTERISTICS

- *Easy to polish*
- *Shades A1, A2, A3, A 3,5, B1, B3*
- *Self-mixing system* in 50ml (76g) cartridges (4:1 ratio) for *optimal dosing and time saving*
- *Optimal working time* (1 min)
- *Quick setting time* (5 min)

acrytemp



GPLCOMMUNICATION.COM



Packaging

C700201 ACRYTEMP A1	1x50ml (76g) cartridge (4:1) + 15x Mixing tips 4:1
C700200 ACRYTEMP A2	1x50ml (76g) cartridge (4:1) + 15x Mixing tips 4:1
C700215 ACRYTEMP A3	1x50ml (76g) cartridge (4:1) + 15x Mixing tips 4:1
C700205 ACRYTEMP A3,5	1x50ml (76g) cartridge (4:1) + 15x Mixing tips 4:1
C700211 ACRYTEMP B1	1x50ml (76g) cartridge (4:1) + 15x Mixing tips 4:1
C700210 ACRYTEMP B3	1x50ml (76g) cartridge (4:1) + 15x Mixing tips 4:1
C700220 ACRYTEMP kit	1 Acrytemp B3 + 1 Acrytemp A2 + 1 Dispenser D2 (4:1) + 30 Mixing tips 4:1

Accessories

C700230	1 x D2 dispenser
U100210	1 x plunger for D2 dispenser
C700240	45x Mixing tips (4:1)

Branch
Zhermack GmbH Deutschland
 Öhlmühle 10 - D-49448 Marl
 Tel. +49 - 0 54 43 / 20 33 - 0
 Fax +49 - 0 54 43 / 20 33 - 11
 info@zhermack.de
 www.zhermack.com

Branch
Zhermapol
 ul. Augustówka 14
 02 - 981 Warszawa - Polska
 Tel. +48 - 22 858 82 72 - 858 73 41
 Fax +48 - 22 642 07 14
 zhermapol@zhermapol.pl
 www.zhermack.com

Branch
Zhermack Inc.
 151 Industrial Way East Bldg B
 Eatontown, NJ 07724
 Phone +1 - 732 389 8540
 Fax +1 - 732 389 8543
 info.usa@zhermack.com
 www.zhermack.com

Representative Office
Zhermack UK and Ireland
 21 Balmoral Road
 Melton Mowbray - Leics.
 LE13 1JN
 Tel. +44 - (0)7870 690811
 uk@zhermack.com
 www.zhermack.com

Representative Office
Zhermack Russia
 5/2 Marshala Vasilevskogo St.
 123098 Moscow - Russia
 Tel. +7 - 0916 9301191
 info.russia@zhermack.com
 www.zhermack.com

Zhermack SpA
 Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY
 Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com

Zhermack®
 BEYOND INNOVATION