discovery®

discovery®
smart

Gebrauchsanweisung / Instructions for use /
Mode d'emploi / Modo de empleo / Modalità d'uso

D
DENTAURUM

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Instruction for use discovery® / discovery® smart brackets

Dear Customer

Thank you for choosing this high-quality product from Dentaaurum.

In order to achieve the best results with this product, it is important to carefully study and follow these instructions for use.

The written instructions cannot cover all possible situations that can occur during use. For this reason, our hotline is available to answer any other questions or concerns that may arise.

Due to constant developments, we recommend, even when using the same products regularly, that you study the enclosed current instructions for use or refer to our website at www.dentaaurum.de.

1. Manufacturer

Dentaaurum GmbH & Co. KG · Turnstraße 31 · 75228 Ispringen · Germany

2. General description

discovery®/discovery® smart are part of the Premium bracket line in the product range of Dentaaurum metal brackets, created from the sum of many years of experience in the field of brackets. discovery®/discovery® smart are constructed as a one-piece bracket and manufactured in the MIM technique – metal injection molding – using a special dental alloy and most modern machines. Per CAD (computer aided design) the brackets are perfectly well adapted to the shape of the tooth and offer ideal handling. The patented laser-structured base ensures the optimal bond and guarantees easy debonding.

3. Working stages

3.1. Patient consultation

Inform the patient about the possible consequences and risks involved in treatment using brackets, in particular during debonding.

3.2. Tooth conditioning

Check each tooth thoroughly for any existing damage to the enamel to avoid being “surprised” by damage to the tooth during debonding. Adhere to the bracket adhesive manufacturer's instructions for conditioning the teeth.

3.3. Bracket adhesive selection

discovery®/discovery® smart brackets can be bonded using all commercially available bracket adhesives. We recommend our ConTec, NO-MIX and MIX adhesive and SmartBond®. Adhere to the manufacturer's instructions for the bracket adhesive as well as the Adhesive System Table in our orthodontic catalog.

3.4. Removal and handling

Grip the discovery®/discovery® smart brackets using clamping tweezers (e.g. REF 025-277-00) mesially-distally or on the distal and/or mesial bracket tie-wings, to remove them from the packaging and handle according to the following steps. The patented laser-structured base of the discovery®/discovery® smart brackets should not be touched with the fingers or contaminated in any way, as this could otherwise detrimentally affect the bond.

3.5. Application of the bracket adhesive

Prepare the bracket adhesive according to the manufacturer's instructions. Apply a small amount of adhesive to the center of the bracket base, but enough to cover the entire bracket base. **This guarantees an optimum bond.**

3.6. Placement and alignment of the bracket

After applying the adhesive, place and press the bracket in the usual way onto the tooth. Optimally align the discovery®/discovery® smart bracket with the angulation aid of the clamping tweezers and press it onto the tooth so that the surplus adhesive exudes from the sides of the bracket base.

It is essential to avoid contact with the antagonist due to the risk of abrasion and/or premature bracket loss. If necessary a palatal bite raise can be used e.g. Bite-Bumper™ (REF 705-000-00).

Completely remove the exuded excess adhesive before it has cured. This is particularly important to ensure easy debonding at the end of treatment.

3.7. Ligating the archwires

Once the adhesive has cured (curing time according to the adhesive manufacturer's instructions), ligating can be completed in the usual way using wire or plastic ligatures.

3.8. Bracket debonding

The excellent bonding characteristics of discovery®/discovery® smart brackets means the process of debonding requires special attention: When debonding discovery®/discovery® smart brackets it is advisable to use a Weingart plier (e.g. REF 003-120-00). Remove the arch wire. Grasp the bracket mesio-distally/diagonally on the tie-wings and slightly compress the pliers, making use of the discovery®/discovery® smart bracket debonding channel to carefully remove the base from the tooth and twist toward the left or the right without pulling away from the tooth.

4. Information about the composition

Information about the composition is included in the materials list, see orthodontic catalog or www.dentaaurum.de.

5. Safety information

Risks and adverse effects (allergic reactions) on discovery®/discovery® smart brackets are not known. The product must not be used if there is a known allergy to one or more of the ingredients.

6. Storage and shelf-life

Store discovery®/discovery® smart brackets in closed packaging.

7. Information for single-use products

discovery®/discovery® smart brackets are intended for single use only. Reconditioning of discovery®/discovery® smart brackets that have already been used (recycling) and reuse on patients is not permitted.

Due to the mechanical wear during treatment and debonding neither dimensional accuracy nor bonding characteristics of the brackets can be guaranteed after a recycling process.

8. Additional tips

- Technical information such as torque, angulation, size etc. can be found on the relevant product label or in the catalog information.
- There are no particular precautions to be taken when disposing of discovery®/discovery® smart brackets after treatment.

9. Delivery program

For the complete range of discovery®/discovery® smart products please see our orthodontics catalog.

10. With reference to our quality standards

Dentaaurum ensures a faultless quality of the products manufactured by us. These recommendations are based upon our own experiences. The user himself is responsible for the processing of the products. Responsibility for failures cannot be taken, as we have no influence on the processing on site.

11. Explanation of symbols used on the label

	Order number
	Batch description
	CE Mark MP Class IIa
	Instructions for use available in the internet
	Not for reuse
	Codablock barcode
	Matrix code

Date of information: 12/11
Subject to modifications

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